Retro-Fit Manual

Puritan-Bennett Gas Control Panel





Important Note

Due to unknown factors of the existing equipment, projects requiring Retro-Fit Conversions must be fully investigated by the installer.

It is the responsibility of the installer to determine if the Retro-Fit will function correctly and if additional components are required. Amico is not responsible for any additional components required.

Retro-Fit Instructions

Remove the screws from the Puritan-Bennett Gas Control Panel.



Remove all internal components.



Retro-Fit Instructions


Mount the filler frame to the wall using the appropriate hardware. There are 6 holes provided for mounting the frame.

NOTE: The frame can not be mounted on the wall centered on the existing back box. The retrofit is shipped with the filler frame attached to the control panel. Before you take these apart, push the assembly into the existing back box and position it into place and mark off the corners of the frame on the wall. These lines will be your reference for mounting the frame to the wall.



Install the male elbows into the threaded connections in the back box. Connect the input tube into the appropriate fitting in the box as shown.

Push the whole assembly gently into the back box and secure the panel to the filler frame using the #8 stainless steel machine screws provided with the kit. Please note the position of the pigtails.

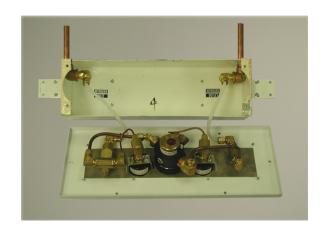

www.amico.com

Amico Pipeline | www.amico.com

85 Fulton Way, Richmond Hill Ontario, L4B 2N4, Canada

71 East Industry Court, Deer Park NY 11729, U.S.A

Toll Free Tel: 1.877.462.6426 Toll Free Fax: 1.866.440.4986

Tel: 905.764.0800 Fax: 905.764.0862 Email: info@amico.com